

ONE YEAR LIMITED WARRANTY

WARRANTY: The company warrants this product to the original purchaser or gift recipient, to be free from defects in workmanship and materials under normal use and services, for a period of one year from the date of purchase.

LIMITATIONS: ALL WARRANTIES IMPLIED BY STATE OR PROVINCIAL LAW, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE EXPRESSLY LIMITED TO THE DURATION OF THE LIMITED WARRANTIES SET FORTH ABOVE. Some jurisdictions do not allow limitation on how long an implied warranty lasts, so the above limitation may not apply to you. IN NO EVENT SHALL THE COMPANY BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES, LOSS OF PROFIT OR MEDICAL EXPENSES CAUSED BY ANY DEFECT, FAILURE, MISUSE, OR MALFUNCTION OF THE PRODUCT. Some jurisdictions do not allow the exclusion of limitation by incidental or consequential damages, so the above limitation or exclusion may not apply to you. The company will not be responsible for damages or losses, direct or indirect, caused by misuse, abuse, accident, negligence, conditions of transport or storage, or failure to follow instruction. The company will not be responsible for any statements that are made or published, written or oral, that are inconsistent with this written warranty, or which are misleading or inconsistent with the facts as published in the literature or specifications furnished by the Company. This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

Warranty Claims Procedure: Any product that proves defective within the warranty period will, transportation charges pre-paid, be repaired or replaced at the option of the Company, free of charge.



Therapeutic Pet Bed Therapy System

Read the contents of this booklet
carefully before product use

Caution:
This product is not suitable for
animals that chew electrical cords.



Phone 1-800-975-0253 if you have questions not answered in these instructions.

INSTRUCTIONS FOR USE - Please read carefully

Congratulations on your Thermotex Therapeutic Pet Bed purchase. Your pet will receive a gentle radiant heat that will keep it warm and deliver therapeutic benefits for many muscle and joint conditions. Don't forget to tell all your animal loving friends about the Thermotex line of Therapeutic Pet Beds.

You will find that after as little as two weeks use, the older animal will gain agility and be more playful as the therapeutic heat helps your pet with conditions such as arthritis. Initially, you may find that your pet spends more time in the bed as well due to the great level of comfort that they receive.

HOW TO USE YOUR PET BED

Your Pet Bed is very simple to use.

Simply plug the bed into any 110 volt outlet and turn the switch to the LOW setting. That is all you have to do. The Pet Bed uses very little power and will cost around 7 cents on the low setting for a 24 hour period.

THERMOTEX PET BEDS ARE NOT TO BE USED ON THE HIGH SETTING.

The low setting provides the best therapeutic benefit. It may take your pet a couple of days to get used to using the new Therapeutic Pet Bed. This is normal.

WASHING INSTRUCTIONS

The cloth liner is washable as is the bed itself. Before washing or dry cleaning the bed you must remove the electrical pad by opening the convenient Velcro closure, and removing the heat insert.

Make sure the Velcro opening is closed before washing. Pet Bed should be washed in lukewarm water and left to air dry. Do not use automatic dryer. Place heat insert back into the bed with the stamped, hard side facing up.

Heat Tolerance

Most animals have different heat tolerances and you will need to experiment with the bed to see what your pet likes best. Always start on the low setting only with no extra lining inside the bed other than the liner that comes with the bed. If your animal finds the bed a bit warm, add an additional liner about the same weight as a regular bath towel. This will lower the surface temperature of the bed making it suit your pet but will not alter the therapeutic effect of the bed. You will soon become aware of your pet's preference.

IMPORTANT SAFETY INSTRUCTIONS

DANGER – to reduce the risk of burns, electric shock and fire, this product must be used in accordance with the following instructions:

- Do not immerse the bed in water unless the heat insert has been removed.
- Do not use pins or other metallic objects to hold bed cover in place.
- Never use the heat insert without outer cover in place.
- Periodically examine the heat insert (PVC). Return for repair if the heat insert shows any sign of deterioration.
- Do not fold or bend the heat insert.
- Do not pull the Pet Bed by the cord or use the cord as a handle.
- Do not use with any type of liniment, salve or ointment.
- Pets should always have the option to move off the bed if desired.
- Do not use on the High setting.

Here are the kinds of comments pet owners are saying about our Therapeutic Pet Beds.

- ◆ The bed is like a day at the spa. She is very relaxed and is much more mobile.
- ◆ My cats and French bull dog loved the heat!
- ◆ My dog seemed more relaxed after using the bed.
- ◆ My pet became more active after using the bed.
- ◆ My older cat became more active and playful after using the bed. She was able to jump more easily.
- ◆ My cat warmed both sides and even her back with the bed and loved it! Very Zippy after using the bed.
- ◆ Our pet is much more nimble when he rises from the bed.
- ◆ My cat used the bed constantly and I noticed that she was not limping as she did prior to using the bed.
- ◆ Our pet literally lived in the bed and used it almost constantly.
- ◆ Our 20 year old cat has trouble moving and after using the pet bed, was jumping up on the furniture again!
- ◆ Our pet loved the bed and actually was snoring when she slept.
- ◆ My 16 and 15 year old cats loved the bed and literally used it 24/7.

We would appreciate photos or comments about our therapy beds.
Please send any information to sales@thermotex.com